

# **Part III Patient Encounters and Performance Skills (PEPS®) Examination**

## **SITE INFORMATION & EQUIPMENT LIST**

**As of February 27, 2024**

The information listed in this Part III PEPS Examination Site Information and Equipment List is for informational purposes only. There may be additional equipment or supplies on-site during the exam that may not be reflected in this manual. **Additionally, consumable supplies (e.g., alcohol wipes, tissues, cotton-tipped applicators) may change without warning.**

This information is not intended to prepare candidates for the exam, but to provide a general overview of equipment and supplies used during the exam. Candidates are reminded to thoroughly review the evaluation forms and Candidate Guide to adequately prepare for the Part III PEPS Examination.

### **Part III PEPS EQUIPMENT QUICK REFERENCE:**

#### **All Stations:**

- Candidate device (Chromebook); candidate will bring into each room

#### **Patient Encounter Stations:**

- Reliance 6200-L Chair
- Optional docking station (desktop monitor, keyboard and mouse)
- Note board (Tagau LCD Writing Tablet 15 inch)

#### **Anterior Segment Station:**


- Reliance 6200-L Chair
- 7900-IC Floor Stand
- Haag-Streit 900 BI Slit Lamp Video System
- Goldmann applanation tonometer with tonometer prism
- Volk 4-Mirror Glass Gonio Fundus Lens with Flange

#### **Posterior Segment Station:**


- Reliance 6200-L Chair
- 7900-IC Floor Stand
- Haag-Streit 900 BI Slit Lamp Video System
- Keeler BIO Vantage Plus Slimline Wireless Plus Indirect System with +2.00 D Oculars
- Volk 20D Condensing Lens
- Volk VDGTL1 Digital High Mag Imaging Lens
- Volk Digital Wide Field Lens
- Volk 78D & 90D Lenses

# CANDIDATE DEVICE

Candidates must log into the device when directed. Candidates will have the device during orientation and throughout the exam. Please review the tutorial for specific instructions on using the platform.

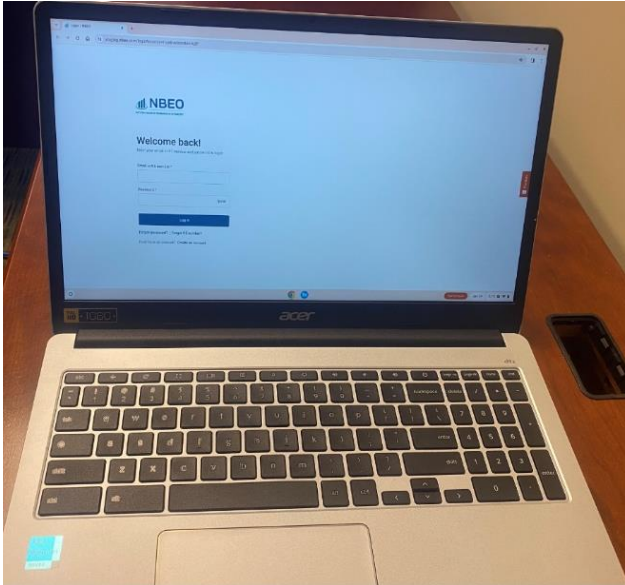

Be mindful of the power button on the device. Candidates must never power down the device during the exam.

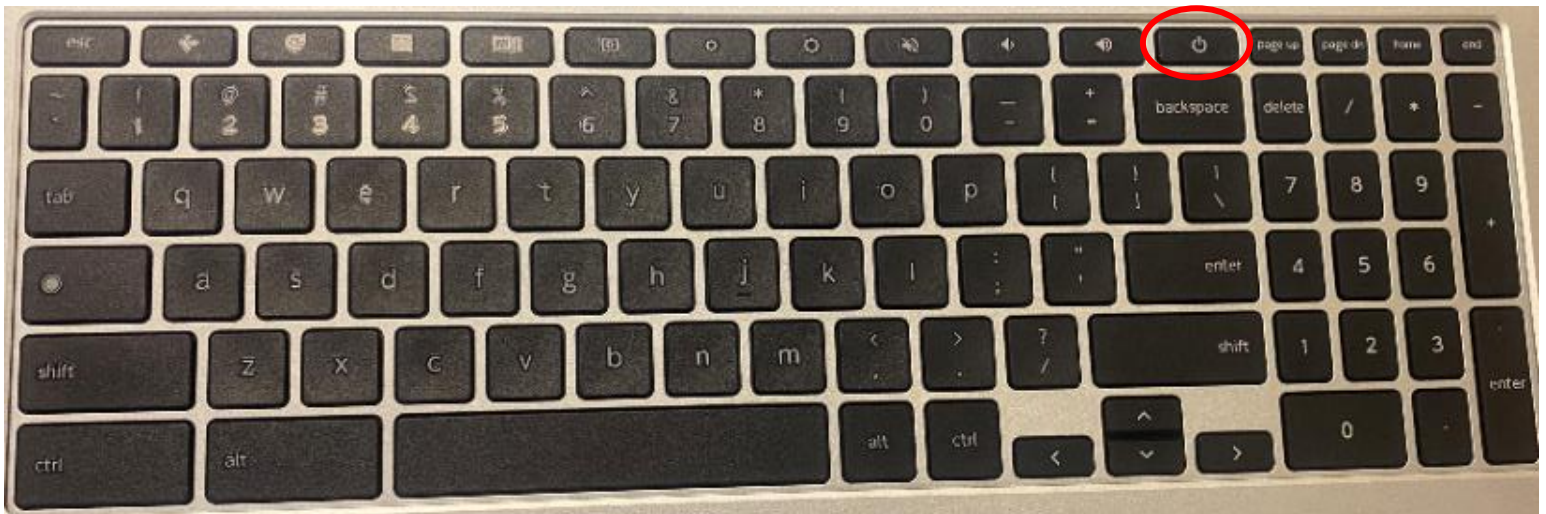

Candidates will have access to the following on their device during orientation time: evaluations, candidate guide, and site information and equipment list. Please note that 7 minutes before the exam is scheduled to start, the candidate's device will no longer be able to access this information.

Candidates will have access to the orientation video and tutorial during orientation but not on the candidate device.

During the exam, the candidate will be able to use their device to:

- View station overview information
- View patient encounter clinical data
- Order and review ancillary tests
- Complete assessment and plans
- Monitor break, observation/overview, and exam cycle time

**PATIENT ENCOUNTER (PE) STATIONS**  
**ROOM OVERVIEW**  
(WITH CANDIDATE DEVICE SHOWN)





# PATIENT ENCOUNTER (PE) STATIONS

## DOCKING STATION

Each PE Station includes a docking station. A docking station provides access to a desktop monitor, keyboard and mouse. The desktop monitor can be used as an extra screen to view images.

**Use of the docking station is optional.**

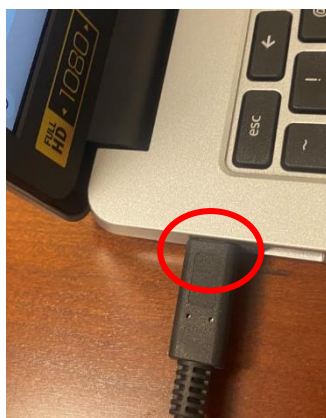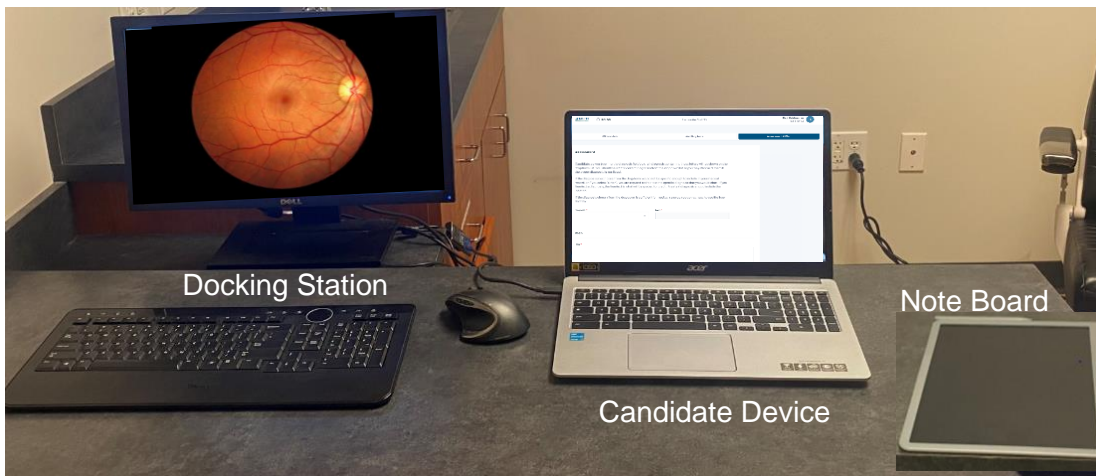

If a candidate wishes to use the docking station, candidates will need to connect the device to the docking station. When connecting to the docking station, ensure the cable is fully pushed into the port as displayed in the image to the left. If you have difficulty connecting, inform the SP or the hallway monitor.

**When connected to the docking station, candidates may use a keyboard/mouse and their laptop screen will be extended onto a desktop monitor. Candidates may drag pictures onto the desk top monitor, as desired.**


# PATIENT ENCOUNTER (PE) STATIONS

## NOTE BOARD

Candidates may take notes in the PE rooms during observation time and throughout the exam cycle using a note board.


**Note boards must remain in the exam room.**

The note board uses a stylus to write. Slide the stylus to the left to use.

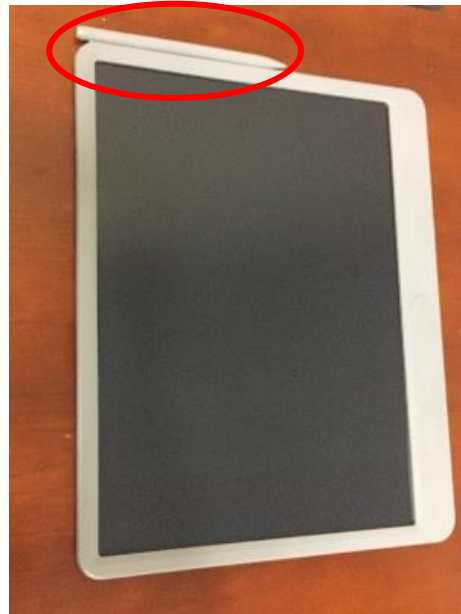

### Lock and Unlock

The note board will be in the locked position when the candidate enters the room. The locked position will prevent the candidate from accidentally clearing their notes. If a candidate wishes to clear their notes, they must unlock it. The unlocking toggle switch is on the back of the device.

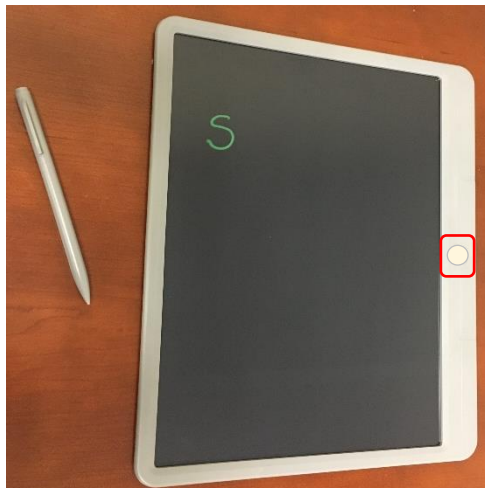

### Clear

To clear out all data on the board, unlock the board and press the button on the right.

Once cleared, the previous information can not be restored.

# SKILLS STATIONS LIGHTING

## MAIN OVERHEAD LIGHT SWITCH, DIMMER LIGHT SWITCH, AND SLIDER

Located to the left of counter area  
Location of this switch is the same in EVERY exam room

Main overhead light switch is located on the LEFT  
Dimmer light switch is located on the RIGHT  
Slider to adjust dimmer brightness is on the RIGHT side of the dimmer switch




Because there are two main overhead light switches in the exam room, candidates may need to place the switch in the up or down position to adjust the overhead lights.



Slider to adjust dimmer brightness

## UNDER-CABINET LIGHT SWITCH

Located above the sink counter, by soap dispenser

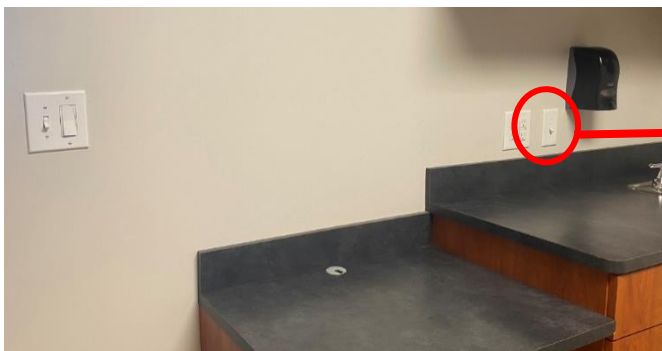


# SKILLS STATIONS

## EXAM CHAIR

### Exam Chair Overview

Reliance 6200-L Chair

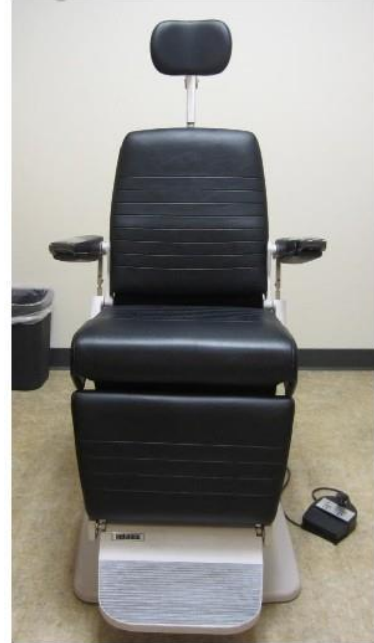



## Exam Chair Overview


### Reliance 6200-L Chair

#### **To recline the exam chair:**

Pull DOWN on the lever located on the back of the chair

Lower the chair and position appropriately

Release the lever while lowered to lock the chair in place

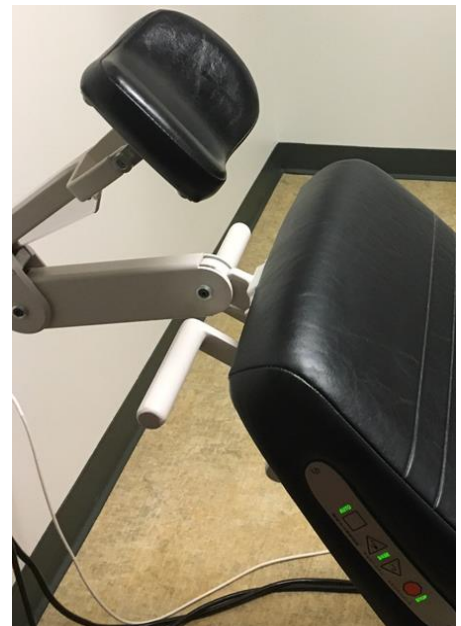

To rotate the chair, push the silver handle towards the floor. When the chair is in position, pull handle to the original position to lock in place.

# SKILLS STATIONS

## SLIT LAMP OVERVIEW


### Slit Lamp

Haag-Streit 900 BI Slit  
Lamp Video System


**CAMERA IS ON THE LEFT OCULAR**

For optimal views,  
candidates must use  
focus rod to focus the  
oculars

**Bright White:**  
Top pin fixed all the way  
to the left,  
**cannot be moved**

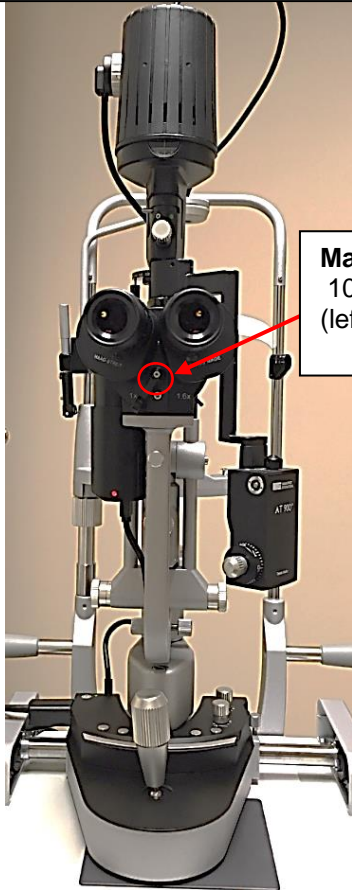


**Magnification:**  
10X or 16X  
(left) (right)

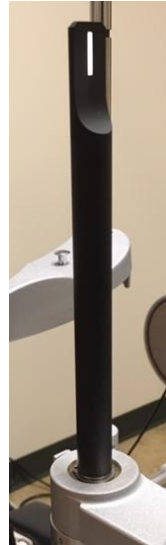


**Power:**  
blinking green = **standby**  
solid green = **on**

**Non-functioning buttons**

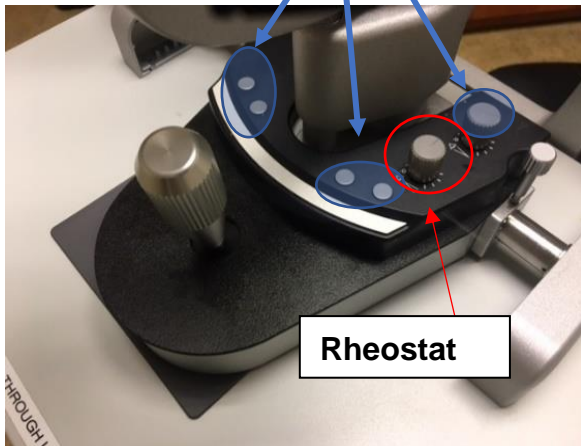

**Rheostat**

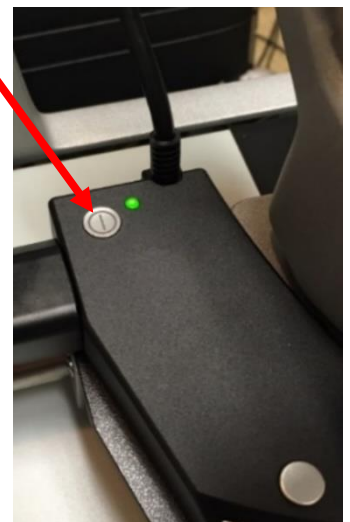



### Blue Filter

Circular dial turned all the way to the **left** so "B" is displayed



### Yellow Filter

If the lever is at , the filter is IN

If the lever is at , the filter is OUT

(the filter cannot be in between these positions)



# ANTERIOR SEGMENT SKILLS STATION

## ROOM OVERVIEW



# ANTERIOR SEGMENT SKILLS STATION


## ADDITIONAL EQUIPMENT AND SUPPLIES

Example of anterior segment station supplies set up on the counter ready for use



### Tray

- Tray will contain ready to use, already-disinfected, gonio lens and tonometer probe
- Please keep tray closed until the start of the exam
- After using the instruments, please place on a clean paper towel

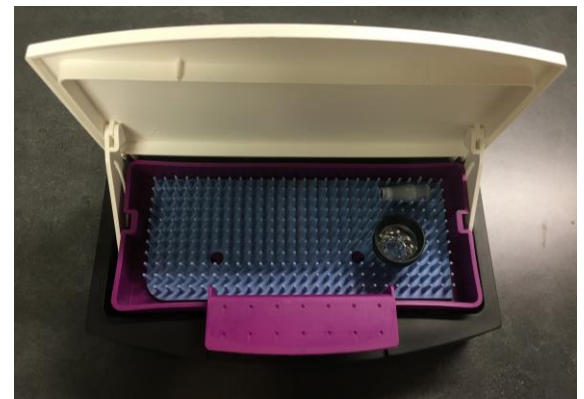

### Gonio Lens

Large Volk 4-Mirror Glass  
Gonio Fundus Lens with Flange



# ANTERIOR SEGMENT SKILLS STATION

## ADDITIONAL SUPPLIES

### Repeat Policy

Posted on the back of the **skills station** exam room door



### Miscellaneous Supplies


Alcohol Wipes



Kim Wipes



Tissues

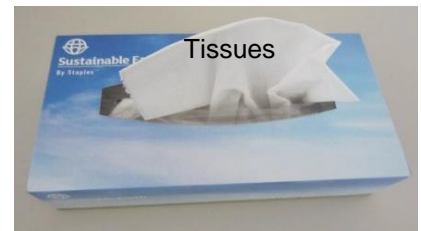


Saline Solution



Cotton Tip Applicators



Elbow Rest

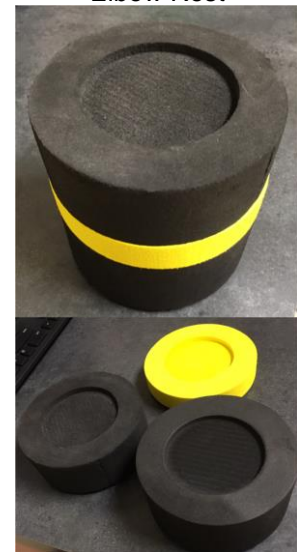

# POSTERIOR SEGMENT SKILLS STATION

## ROOM OVERVIEW



# POSTERIOR SEGMENT SKILLS STATION

## BIO

### Keeler BIO Vantage Plus Slimline Wireless Plus Indirect System

- The BIO headset will be hanging on the stand
- Without an appropriate target and working distance (18-25 inches) the target will appear decentered
- The BIO has been equipped with our camera system to ensure captured views




#### FIXED

- aperture
- filter
- brightness
- circle beam height
- +2.00 oculars

#### ADJUSTABLE

- fit behind head
- fit on top of head
- PD
- Pantoscopic tilt: angle of ocular to eyes

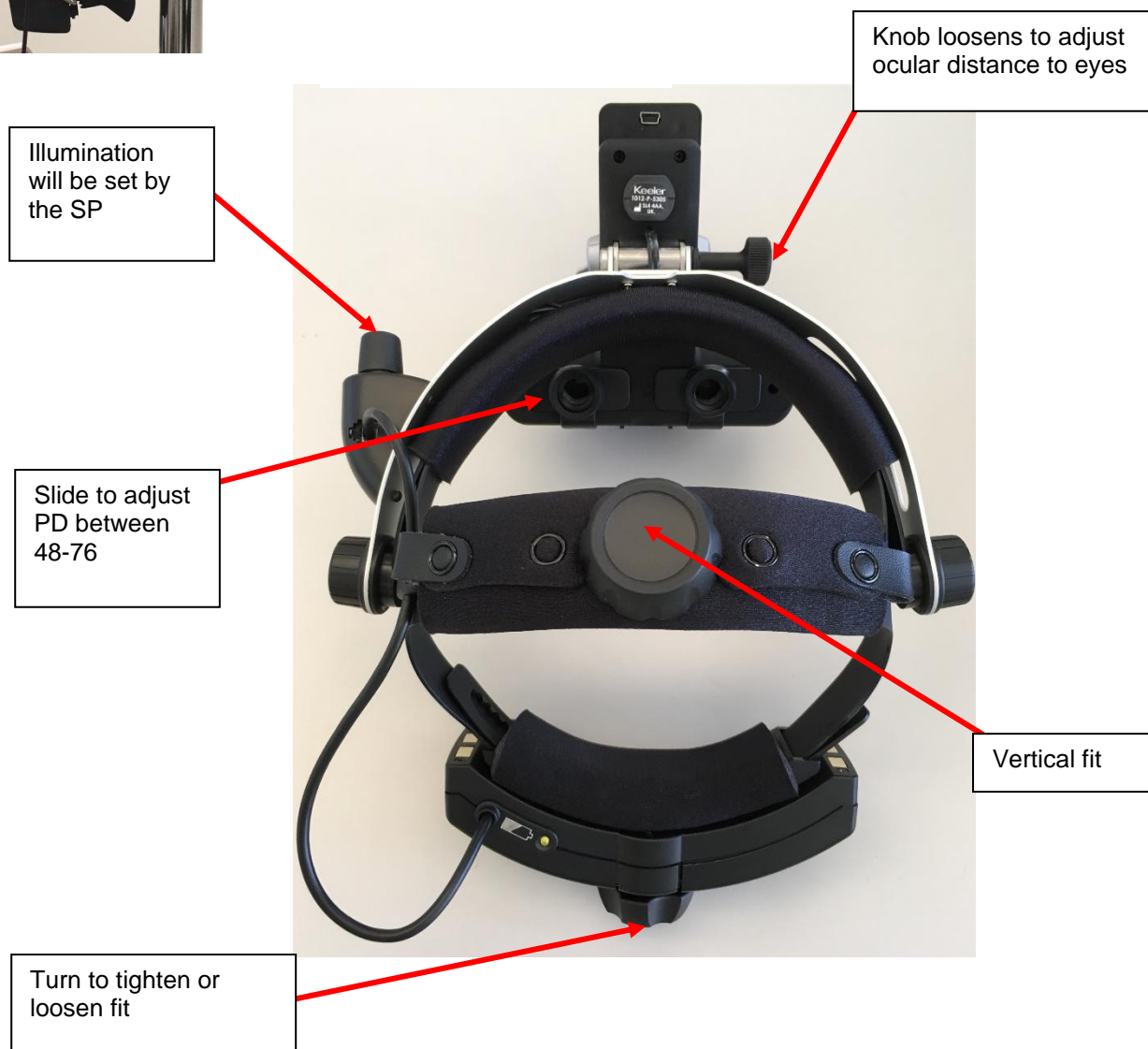



# POSTERIOR SEGMENT SKILLS STATION

## ADDITIONAL EQUIPMENT AND SUPPLIES

### Lens Overview

The following lenses are available for candidate use during posterior evaluation:

- Volk 20D Lens
- Volk VDGL1 Digital High Mag Imaging Lens
- Volk Digital Wide Field Lens
- Volk 78D & 90D Lenses



### Miscellaneous Supplies

Alcohol Wipes



Kim Wipes




Tissues



### Repeat Policy

Posted on the back of the **skills station** exam room door

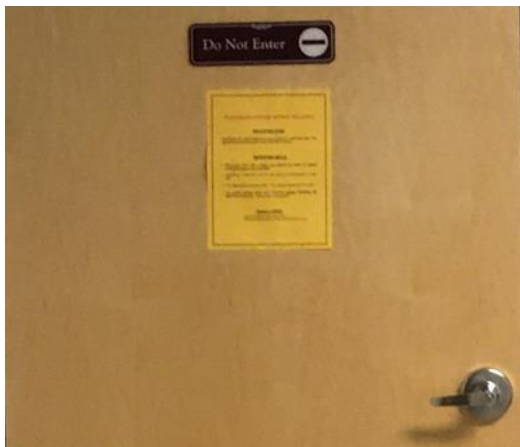

Elbow Rest



**NBEO would like to extend its collective best wishes for your success.**